

The collector/administrator is the person who records and administers the test.


1. Open a new drug test device and place on the counter.
2. Have a clean, dry specimen cup ready for the urine sample. Preferably use a [specimen cup](#) that has a temperature strip on the side so you can check the temperature. (For dip cards only)
3. Check the sample temperature. The temperature should be within 93° to 98° Fahrenheit. If it's out of range, ask the donor to provide a new sample or ask them to wait until they can.
4. Do not allow them to leave the area until the test is complete.
5. Examine the urine color; it should be light yellow with an odor; fake urine has no odor. Bright yellow or clear are warning signs that the sample may have been substituted or diluted.
6. We recommend that you use a [Drug adulteration test strip](#) whenever possible to establish the integrity of the sample.

Instructions to the donor. (The donor is the person providing the urine for testing).

1. Ask them to remove any outer garments as they may be concealing paraphernalia and place on one side.
2. Then ask them to empty their pockets and turn them out. This ensures they are not hiding anything.
3. Now ask them to wash their hands to clean off any adulterants that may be present on their hands or under their fingernails.
4. Tell the person they need a minimum urine sample of 30 mL and to complete the task as quickly as possible.
5. Tell them not to flush the toilet after completion. You need to check the bathroom for signs of discarded paraphernalia.

Dip test instructions.

1. Open the package containing the drug test.
2. Discard the desiccant in the package.
3. Remove the protective cap from the dip drug test and immerse only the tips into the urine sample. Do not flood the device as this will invalidate the test.
4. Remove from urine sample after 20 seconds.
5. Replace the protective cap and place device on a flat surface.
6. Within the first minute, you will start to see a reaction with lines gradually appearing.
7. Read the result at 5 minutes.

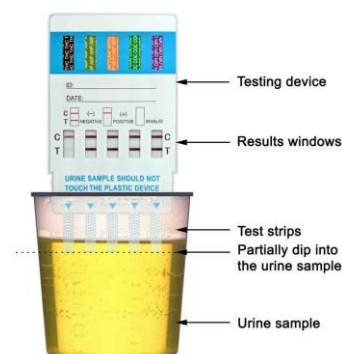

Cup test instructions.

1. Open the package containing the drug test.
2. Discard the desiccant in the package.
3. Remove the lid on the cup and ask the donor to provide a sample directly into the container to the level marked on the side.
4. Replace the top and check the temperature.
5. Depending on the type of cup used. Always refer to the device instructions.
6. Read the result at 3 minutes.

Interpreting the results

