
FENTANYL TEST STRIPS

Intended For Harm Reduction


Hold this end




USER INSTRUCTIONS FOR TESTING NON-OPIOID CRYSTAL OR POWDERED SUBSTANCES

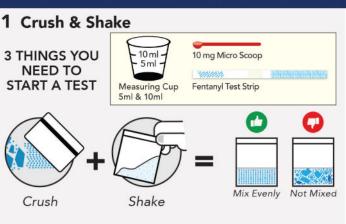


A visual guide to testing only a portion of your substance. This method should only be used if you cannot dilute and test everything.

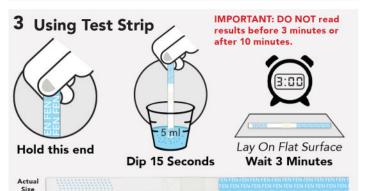

Do not insert beyond dotted area

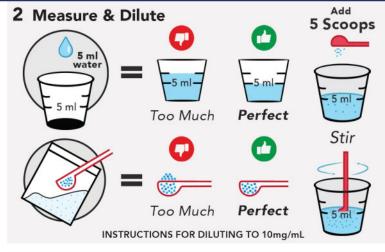

Interpreting The Results

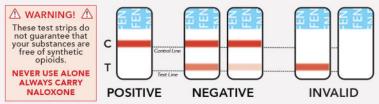

One red line on top after waiting three minutes is a POSITIVE result for the presence of fentanyl.

Two red lines is a **NEGATIVE** result. The lower red line may be significantly lighter than the upper red line. If you can see it at all after waiting three minutes, no matter how faint, it is still a negative result.

No red lines (or one red line on the bottom) means the test is invalid. Usually this happens because the liquid did not travel far enough up the strip. Retest with new strip.